

Natrelle®



Offering a
SMOOTH EXPERIENCE
from the start

Breast
AUGMENTATION

Breast
RECONSTRUCTION

Actual Natrelle INSPIRA® patients.
Individual results may vary.

*Based on surgeon survey data, September 2020 (N = 118).

Please see inside for Indications and Important Safety Information.

The right implant depends on your PATIENTS' NEEDS

When patients are considering breast implants, some things that come to mind include the look, the feel, and, of course, safety. Since every patient has different needs and wants, we're proud to offer the largest smooth portfolio. It'll help pave the way for a smooth experience for you and your patients, from start to finish, and beyond.



Natrelle® Breast Implants Important Information

INDICATIONS

Natrelle® Breast Implants are indicated for women for the following:

- **Breast augmentation for women at least 22 years old for silicone-filled implants. Breast augmentation for women at least 18 years old for saline-filled implants.** Breast augmentation includes primary breast augmentation to increase breast size, as well as revision surgery to correct or improve the result of a primary breast augmentation surgery.
- **Breast reconstruction.** Breast reconstruction includes primary reconstruction to replace breast tissue that has been removed due to cancer or trauma or that has failed to develop properly due to a severe breast abnormality. Breast reconstruction also includes revision surgery to correct or improve the result of a primary breast reconstruction surgery.

IMPORTANT SAFETY INFORMATION

CONTRAINDICATIONS

Breast implant surgery should not be performed in:

- Women with active infection anywhere in their body.
- Women with existing cancer or precancer of their breast who have not received adequate treatment for those conditions.
- Women who are currently pregnant or nursing.

WARNINGS

- Breast implants are not lifetime devices or necessarily a one-time surgery.
- **Avoid damage during surgery:** Care should be taken to avoid the use of excessive force and to minimize handling of the implant. Use care when using surgical instruments in proximity with the breast implant. For more information, please see the full Directions for Use.

PRECAUTIONS


Safety and effectiveness have not been established in patients with the following:

- Autoimmune diseases (eg, lupus and scleroderma).
- A compromised immune system (eg, currently receiving immunosuppressive therapy).
- Planned chemotherapy following breast implant placement.
- Planned radiation therapy to the breast following breast implant placement.
- Conditions or medications that interfere with wound healing and blood clotting.
- Reduced blood supply to breast tissue.
- Clinical diagnosis of depression or other mental health disorders, including body dysmorphic disorder and eating disorders. Please discuss any history of mental health disorders prior to surgery. Patients with a diagnosis of depression, or other mental health disorders, should wait until resolution or stabilization of these conditions prior to undergoing breast implantation surgery.

Please see additional Natrelle® Breast Implants Important Safety Information on page 4.

The desired look your PATIENTS WANT

19% **61%** **11%** **9%**
of prospective **AUGMENTATION** patients surveyed preferred this look.^{2,*}

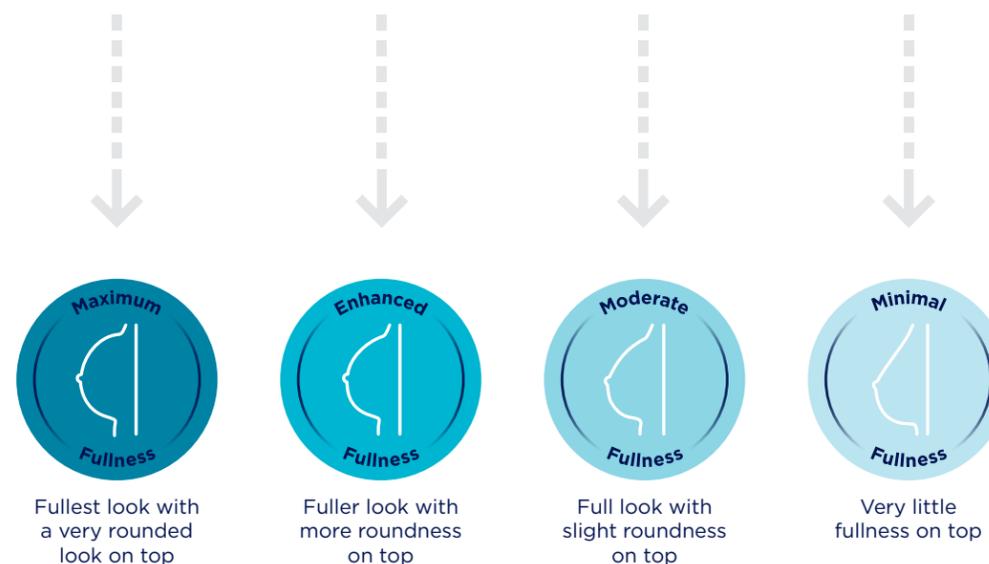

15% **36%** **42%** **7%**
of prospective **RECONSTRUCTION** patients surveyed preferred this look.^{3,†}



*Based on a 2017 survey of 141 prospective breast augmentation patients.

†Based on a 2020 survey of 98 prospective and existing breast reconstruction patients.

Actual Natrelle INSPIRA® patients. Individual results may vary.

Natrelle® offers 5 PROFILES for every patient type⁴

With the *Natrelle* INSPIRA® Smooth Collection, you can shape the breasts of all your patients—whether it’s reconstruction, revision, or augmentation.



The *Natrelle*® Portfolio also includes saline-filled breast implants.



Actual *Natrelle* INSPIRA® patient. Individual results may vary.

Natrelle® Breast Implants IMPORTANT SAFETY INFORMATION (continued) ADVERSE EVENTS

Key adverse events are reoperation, implant removal with or without replacement, implant rupture with silicone-filled implants, implant deflation with saline-filled implants, and capsular contracture Baker Grade III/IV.

Other potential adverse events that may occur with breast implant surgery include: asymmetry, breast pain, breast/skin sensation changes, capsular calcification, delayed wound healing, hematoma, hypertrophic scarring/scarring, implant extrusion, implant malposition, implant palpability/visibility, infection, nipple complications, redness, seroma, swelling, tissue/skin necrosis, wrinkling/rippling.

For more information see the full Directions for Use at www.allergan.com/products. To report a problem with *Natrelle*® Breast Implants, please call Allergan at 1-800-433-8871.

Natrelle® Breast Implants are available by prescription only.

Three cohesivities. COUNTLESS POSSIBILITIES.^{4,*}

Only *Natrelle* INSPIRA® offers **3 different cohesivities⁵**

Smooth implants designed to retain upper pole fullness.†


How well an implant maintains its fullness when held upright and keeps its overall shape depends on its cohesivity. An implant with highly cohesive gel will hold its shape more than an implant with responsive gel.^{6,†}



*300 smooth breast implant options.
†Based on implant performance testing; clinical significance has not been established.

Images for representation only. In vivo significance has not been established.

Smooth implants designed to hold their shape.†

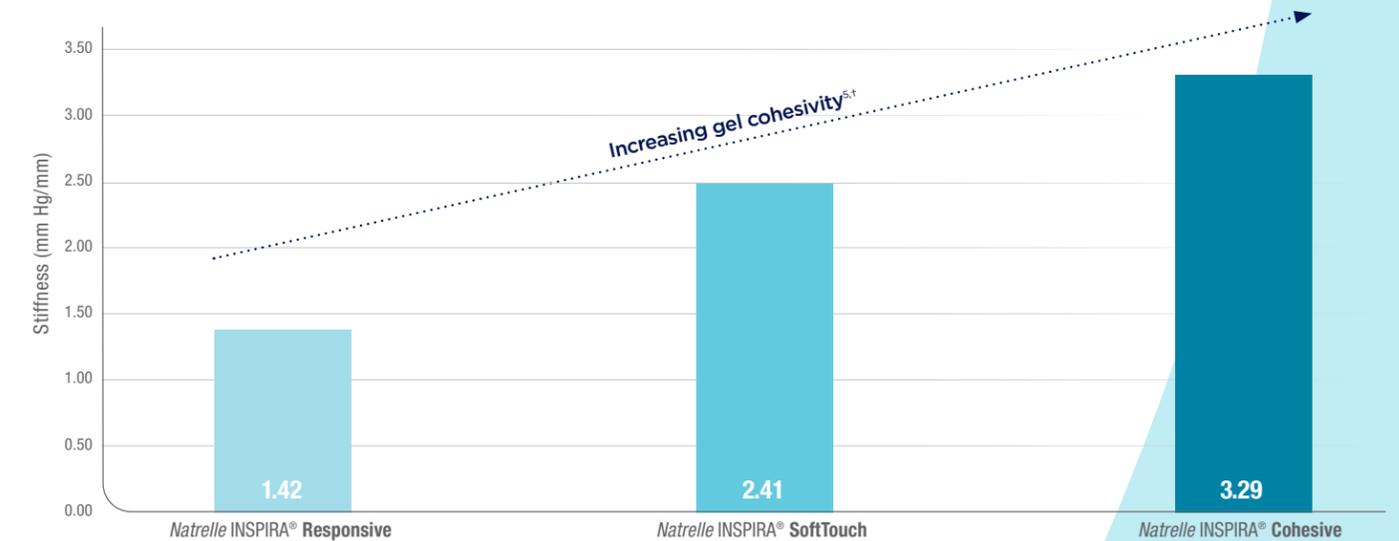

Natrelle INSPIRA® Collection

#1 selected round gel implant collection in the US^{1,5}

†Methodology Breast implant devices (n = 6 per group) were placed in a horizontal orientation on a sliding stage; the width and maximum projections of the implants were measured using fixed calipers. The devices were then placed in a vertical-supported orientation using a 90° angle, and the width and maximum projections were measured again. From those measurements, the retention of dimension was calculated and the relative change was determined.

⁵Based on surgeon survey data, September 2020 (N = 118).


ADVANCED implant technology

What's in our implants matters. When you and your patients are considering *Natrelle*®, be sure to tell them about the unique composition of *Natrelle* INSPIRA® Breast Implants, because it may help address some common concerns they have.

A gel designed to stay in place.

The cohesive gel in all *Natrelle*® gel breast implants is designed to hold together. When *Natrelle*® gel implants are cut ex vivo, the gel stays in place.*

INTRASHIEL™ barrier shell technology minimizes diffusion through the silicone elastomer shell.⁷

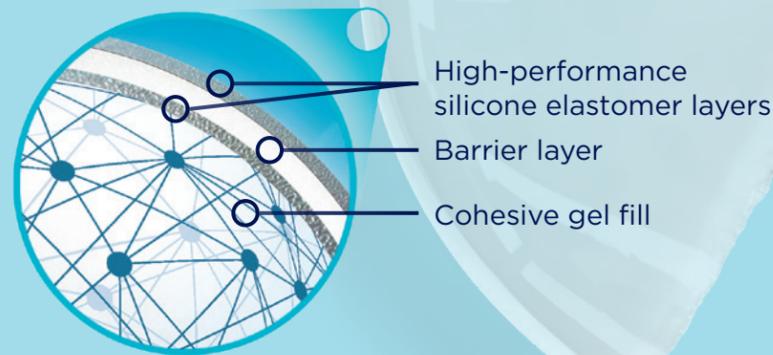

*In vivo significance has not been established.

Natrelle INSPIRA®
Cohesive

The most cohesive round gel breast implant in the US.⁵

Natrelle® 133S Smooth Tissue Expanders With MAGNA-SITE® Injection Sites Important Information

INDICATIONS

Natrelle® 133S Smooth Tissue Expanders are indicated for:

- Breast reconstruction following mastectomy.
- Treatment of underdeveloped breasts.
- Treatment of soft tissue deformities.

IMPORTANT SAFETY INFORMATION

CONTRAINDICATIONS

Natrelle® 133S Smooth Tissue Expanders **SHOULD NOT** be used in patients:

- Who already have implanted devices that would be affected by a magnetic field (eg, pacemakers, drug infusion devices, artificial sensing devices).
- Whose tissue at the expansion site is determined to be unsuitable.
- Who have an active infection or a residual gross tumor at the expansion site.
- Undergoing adjuvant radiation therapy.
- Whose physiological condition (eg, sensitive over- or underlying anatomy, obesity, smoking, diabetes, autoimmune disease, hypertension, chronic lung or severe cardiovascular disease, or osteogenesis imperfecta) or use of certain drugs (including those that interfere with blood clotting or affect tissue viability) poses an unduly high risk of surgical and/or postoperative complications.
- Who are psychologically unsuitable.

WARNINGS

- **DO NOT** use *Natrelle*® 133S Smooth Tissue Expanders in patients who already have implanted devices that would be affected by a magnetic field (see Contraindications) because the MAGNA-SITE® integrated injection site contains a strong rare-earth, permanent magnet. Diagnostic testing with Magnetic Resonance Imaging (MRI) is contraindicated in patients with *Natrelle*® 133S Smooth Tissue Expanders in place.

Natrelle® ConfidencePlus® warranty program



Help your patients make a confident choice.

ConfidencePlus® gel warranty

FREE coverage and automatic enrollment

Rupture

- Implant replacement: Lifetime for both affected and contralateral implants
- Out-of-pocket financial assistance: Up to \$3500 for 10 years

Capsular contracture Baker Grade III/IV

- Implant replacement: 10 years for both affected and contralateral implants
- Out-of-pocket financial assistance: Up to \$2000 for 2 years (augmentation patients only)*

Late seroma

- Implant replacement: 20 years for both affected and contralateral textured implants

Late seroma diagnostic testing coverage

If the patient has textured gel or textured saline breast implants, Allergan will cover up to \$1000 of out-of-pocket fees toward diagnostic testing for late seroma to rule out breast implant-associated anaplastic large cell lymphoma (BIA-ALCL).

BIA-ALCL treatment coverage

In the event of a BIA-ALCL diagnosis, the patient is eligible for up to \$7500 out-of-pocket financial assistance toward the removal of the breast implant(s) and the associated scar tissue (complete capsulectomy).

Allergan will also provide replacement implant(s) at no charge. For coverage eligibility, surgeons are directed to contact Allergan and provide appropriate documentation.

*Covers one incident per patient; recurring capsular contracture is not covered.

Natrelle® 133S Smooth Tissue Expanders IMPORTANT SAFETY INFORMATION (continued)

WARNINGS (continued)

- **DO NOT** alter the tissue expander or use adulterated fill. Fill only with sterile saline for injection as described in INSTRUCTIONS FOR USE. **DO NOT** expose to contaminants.
- **DO NOT** expand if the pressure will compromise wound healing or vasculature of overlying tissue, or beyond patient or tissue tolerance. Stop filling immediately if tissue damage, wound dehiscence, abnormal skin pallor, erythema, edema, pain, or tenderness are observed.
- **DO NOT** reuse explanted products.
- Active infection anywhere may increase risk of periprosthetic infection. **DO NOT** expose the tissue expander or injection needles to contaminants. Postoperative infections should be treated aggressively.
- Adverse reactions may require premature explantation.
- When using suturing tabs be careful to avoid piercing the shell. Use a new one if damage occurs.
- *Natrelle*® 133S Smooth Tissue Expanders are temporary devices and are not to be used for permanent implantation or beyond 6 months. Tissue expansion in breast reconstruction typically requires 4 months to 6 months.

PRECAUTIONS

Active infections may need to be treated and resolved before surgery. Allergan relies on the surgeon to know and follow proper surgical procedures and carefully evaluate patient suitability using standard practice and individual experience. Avoid damage to the tissue expander and use a sterile backup in case of damage. Pay careful attention to tissue tolerance and hemostasis during surgery. Expansion should proceed moderately and never beyond patient or tissue tolerance. Avoid contamination in any postoperative procedure.


ADVERSE REACTIONS

Deflation, tissue damage, infection, extrusion, hematoma/seroma, capsular contracture, premature explantation, displacement, effects on bone, pain, sensation, distortion, inadequate tissue flap, and inflammatory reaction.

For more information, please visit www.allergan.com/products. To report a problem with *Natrelle*®, please call Allergan at 1-800-433-8871.

Natrelle® 133S Smooth Tissue Expanders are available by prescription only.

Expanding PATIENT REACH to your practice

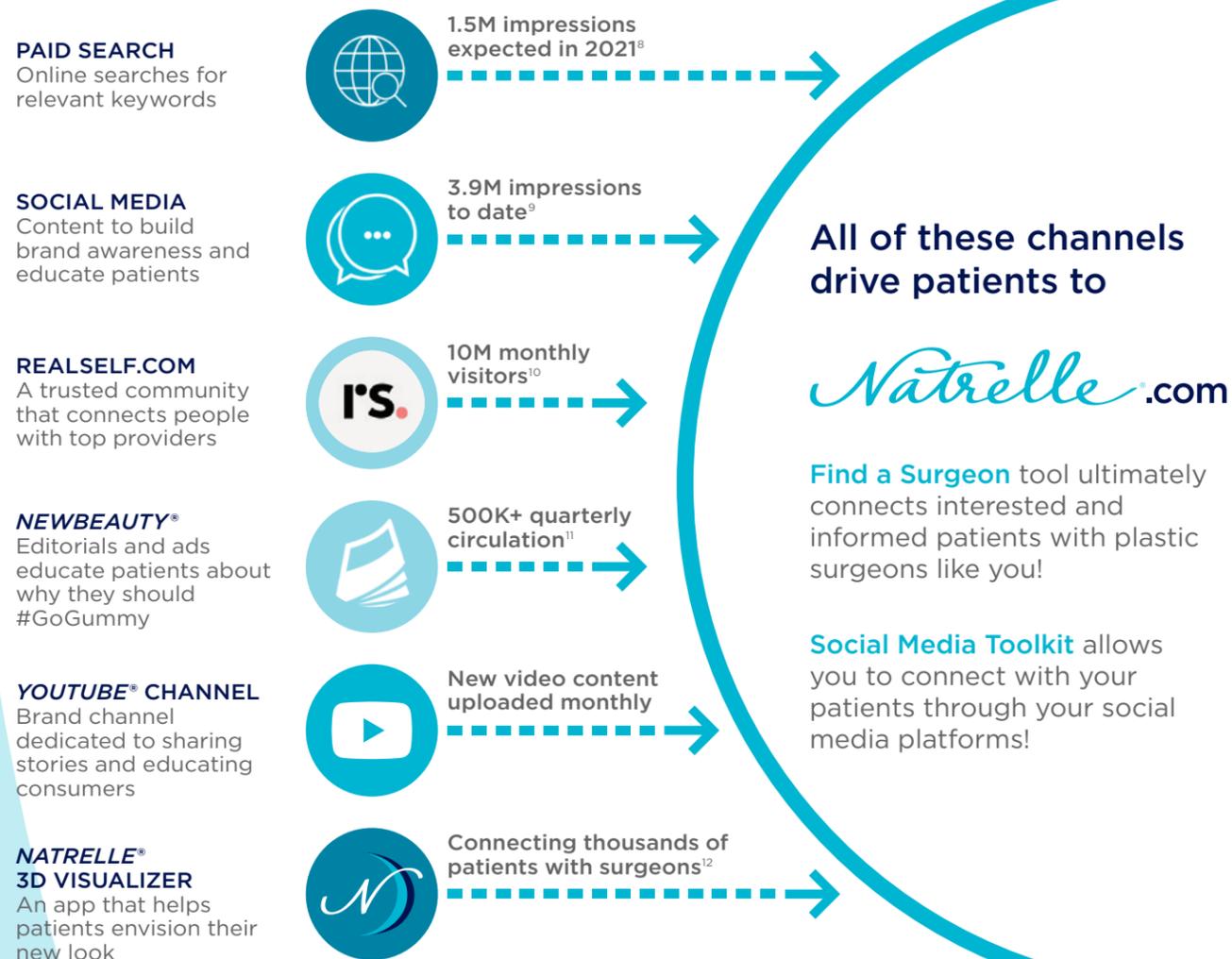

2 exclusive PATIENT OFFERS that benefit your practice



Natrelle® GEL REWARDS

CHOOSE A REWARD

When augmentation patients select *Natrelle*® gel breast implants, they will receive a FREE Allergan Aesthetics treatment or product.*

SHARE WITH A FRIEND

Patients can also give a friend a FREE Allergan Aesthetics treatment or product.*



Introducing AllēSM:

An even more brilliant loyalty program by Allergan Aesthetics.

EARN

Patients earn 500 points with a *Natrelle*® gel breast augmentation.


REDEEM

Patients save on other Allergan Aesthetics treatments and products.

There are 3 ways to help your consumers become an Allē member:

1. Direct them to Alle.com to create an account.
2. Send an Allē invite link to their mobile device via the provider dashboard.
3. Suggest they call Allē member support at 888-912-1572.

Natrelle® Gel Rewards Program encourages repeat patients.

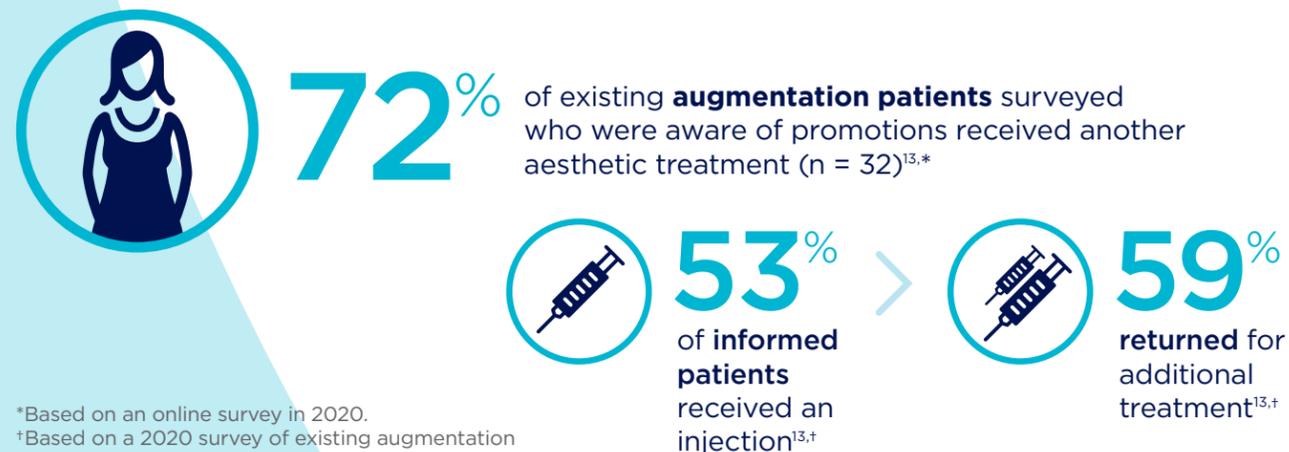

*Based on an online survey in 2020.
†Based on a 2020 survey of existing augmentation patients who were aware of promotions (n = 32) and received an injection (n = 17).

*Offerings are subject to change. Terms and conditions apply; for qualified augmentation patients only. Patient has the ability to choose any physician of her choice.

Women who went gummy

BREAST AUGMENTATION



Revision Augmentation With Breast Lift With *Natrelle* INSPIRA® Cohesive

BEFORE




AFTER



Natrelle INSPIRA® Cohesive Style SCM-445
Implant dimensions: 445 cc volume, 13.50 cm diameter, 4.5 cm projection
Photos provided by Dr Allen Gabriel.
Individual results may vary.

Primary Augmentation With *Natrelle* INSPIRA® SoftTouch

BEFORE

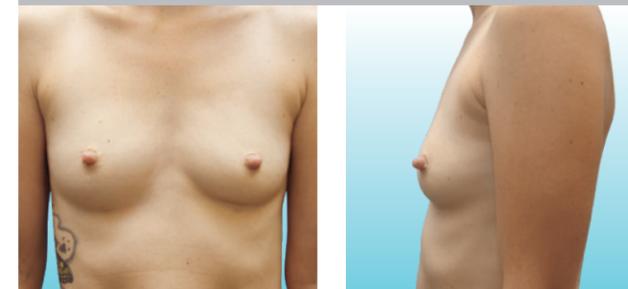

AFTER




Natrelle INSPIRA® SoftTouch Style SSM-310
Implant dimensions: 310 cc volume, 12.00 cm diameter, 4.0 cm projection
Photos provided by Dr Gaurav Bharti.
Individual results may vary.



Primary Augmentation With *Natrelle* INSPIRA® Cohesive

BEFORE

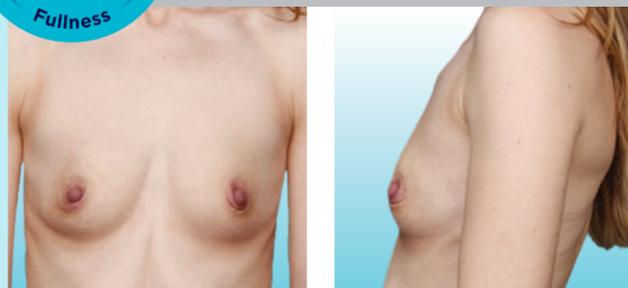

AFTER



Natrelle INSPIRA® Cohesive Style SCX-580
Implant dimensions: 580 cc volume, 13.50 cm diameter, 6.4 cm projection
Photos provided by Dr Allen Gabriel.
Individual results may vary.

Primary Augmentation With *Natrelle* INSPIRA® Cohesive

BEFORE



AFTER



Natrelle INSPIRA® Cohesive Style SCM-310
Implant dimensions: 310 cc volume, 12.00 cm diameter, 4.0 cm projection
Photos provided by Dr Allen Gabriel.
Individual results may vary.



SMOOTH surface.
PRECISE expansion.

Available in
36 smooth options and
6 profiles⁴



Natrelle
133S smooth tissue expander

- Smooth surface for less surface area and less tissue adherence^{14,*}
- 360° tab orientation for greater placement support¹⁵

Natrelle[®] 133S Smooth Tissue Expander features the FOURTÉ[®] Expander Fill System for 4X faster fill than a standard 21-gauge needle.^{16,*,+}

*Clinical significance has not been established.

[†]**Methodology** The FOURTÉ[®] Expander Fill System and 21-gauge needle are attached to 60-cc syringes filled with water. The injector starts the injection at maximum effort while the time is tracked. When the plunger reaches the end of the syringe barrel, the time is stopped and recorded. This is repeated 20 times, recorded, and compared.

PAIRED FOR PRECISION for
the breast shape you envision¹⁸



Natrelle[®] 133S Smooth Tissue Expanders deliver a seamless match between smooth tissue expander and round breast implant.^{17,*}



100% MATCH

between all *Natrelle*[®] smooth tissue expanders and *Natrelle* INSPIRA[®] Breast Implants (in both base width and projection)^{17,*}

Actual patient with *Natrelle*[®] tissue expanders.
Individual results may vary.

^{*}**Methodology** Tissue expanders were matched to round breast implants. A precise match was defined as a breast implant that falls within a range of 0.5 cm smaller to 1.0 cm larger in base width and 1.5 cm smaller to 2.0 cm larger in projection of a tissue expander. The percentage of precise matches was calculated for each style of tissue expander.


Women who chose *Natrelle*® for BREAST RECONSTRUCTION



91% of women who had breast reconstruction with *Natrelle*® breast implants were satisfied with their results at 10 years.^{7*}



Breast Reconstruction With Pre-pec Placement and Fat Transfer

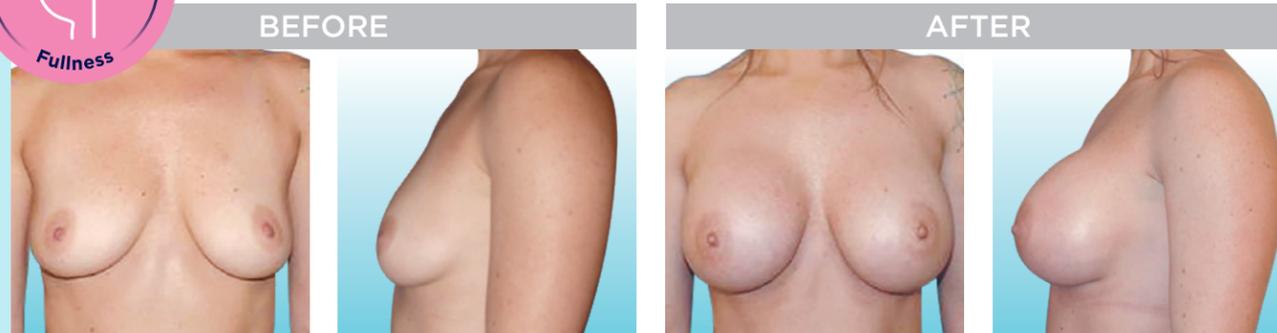

Natrelle INSPIRA® Cohesive Style SCF-650
 Implant dimensions: 650 cc volume, 14.50 cm diameter, 5.9 cm projection
 Photos provided by Dr Ritu Chopra.
Individual results may vary.



Revision Reconstruction With *Natrelle* INSPIRA®



Natrelle INSPIRA® Responsive Style SRL-110 (right)
 Implant dimensions: 110 cc volume, 10.00 cm diameter, 2.0 cm projection
Natrelle INSPIRA® Cohesive Style SCF-415 (left)
 Implant dimensions: 415 cc volume, 12.50 cm diameter, 5.1 cm projection
 Photos provided by Dr Hani Sbitany.
Individual results may vary.



Breast Reconstruction With Pre-pec Placement and Fat Transfer



Natrelle INSPIRA® Cohesive Style SCF-415
 Implant dimensions: 415 cc volume, 12.50 cm diameter, 5.1 cm projection
 Photos provided by Dr Allen Gabriel.
Individual results may vary.



Breast Reconstruction With Pre-pec Placement



Natrelle INSPIRA® SoftTouch Style SSF-520
 Implant dimensions: 520 cc volume, 13.25 cm diameter, 5.5 cm projection
 Photos provided by Dr Maurice Nahabedian.
Individual results may vary.

*Patient satisfaction based on primary reconstruction patients using round implants (n = 43; 67.4% definitely satisfied and 23.3% somewhat satisfied).

A photograph of three women in bikinis standing by a body of water. The woman on the left is wearing a teal one-piece swimsuit. The woman in the center is wearing a black one-piece swimsuit. The woman on the right is wearing a black two-piece bikini. They are all looking down and smiling slightly. The background shows water with sunlight reflecting off it.

Visit [NatrelleSurgeon.com](https://www.natrellesurgeon.com) to learn more.



@NatrelleBreastAugmentation
@NatrelleBreastReconstruction

Actual *Natrelle* INSPIRA® patients.
Individual results may vary.

Your practice
Our purpose

Empowering you to meet more patients' needs motivates every decision we make—fueling our purpose now and beyond.

References: 1. Data on file, Allergan Aesthetics, September 2020. 2. Data on file, Allergan, October 2017. 3. Data on file, Allergan Aesthetics, July 2020. 4. Data on file, Allergan, January 2020. 5. Data on file, Allergan, February 2018. 6. Data on file, Allergan, February 2017. 7. *Natrelle*® Silicone-Filled Breast Implants and *Natrelle* INSPIRA® Breast Implants: Smooth & BIOCELL® Texture Directions for Use, 2017. 8. Data on file, Allergan Aesthetics, October 2020. 9. Data on file, Allergan Aesthetics, November 2020. 10. *RealSelf*® Insights Center. *RealSelf* website. <https://insightscenter.realself.com>. Accessed May 1, 2018. 11. *NewBeauty*® 2018 Media Kit. Sandow website. http://www.sandow.com/wp-content/uploads/2014/07/NB_2018_MediaKit_110118_SL.pdf. Accessed October 25, 2018. 12. Data on file, Allergan Aesthetics, October 2020. 13. Data on file, Allergan, April 2020. 14. O'Shaughnessy K. Evolution and update on current devices for prosthetic breast reconstruction. *Gland Surgery*. 2015;4(2):97-110. 15. *Natrelle*® 133S and *Natrelle*® 133 Plus Tissue Expanders Directions for Use, 2018. 16. Data on file, Allergan, January 2016. 17. Data on file, Allergan, January 2017.

Allergan
Aesthetics
an AbbVie company

NATRELLE® and its design are trademarks of Allergan, Inc., an AbbVie company.
All other trademarks are the property of their respective owners.
© 2020 AbbVie. All rights reserved. Natrelle.com NAT119031-v3 12/20 008703

Natrelle®